

Mamalu pro™

SPECIAL NEEDS STROLLER




ATTENTION

Manufacturer takes responsibility for proper usage of the device only when the product was purchased from qualified Akces-MED Ltd. employee or at specialized medical store.

Akces-MED Ltd. reserve the right to introduce technical and trading changes in the content of the instruction without warning.

Product marked with the CE sign is Class I medical device, of rule 1, consistent with the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices.

Akces-MED Ltd. Company introduced quality management system ISO 13485 regarding production, sales and service.

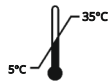

Contents

- Introduction.....04
- 1. Purpose of user manual.....05
- 2. Indications and contraindications for use.....05
- 3. General characteristic of the product.....07
- 4. Size table.....08
- 5. How to use the product.....10
 - 5.1 Unfolding the product.....10
 - 5.2 Folding the product.....11
 - 5.3 Backrest tilt-angle adjustment.....14
 - 5.4 Seat depth adjustment.....15
 - 5.5 Footrest height adjustment.....16
 - 5.6 5-point safety belts.....16
 - 5.7 Rear wheels with independent brakes.....19
 - 5.8 Front swivel wheels with direction blockade.....20
 - 5.9 Push handles.....20
- 6. Use of the stroller.....21
 - 6.1 Preparation of the user.....21
 - 6.2 Getting on and off the product.....21
 - 6.3 Sitting position.....23
 - 6.4 Position when using a special needs stroller.....23
 - 6.5 Driving a special needs stroller.....24
 - 6.6 How to use vests.....25
- 7. Safety rules.....28
- 8. Terms of warranty.....30
- 9. Maintenance and cleaning.....31
- 10. Labelling.....35

Introduction

MAMALU PRO™ – the latest generation of specialized stroller designed to make strolling with your child incredibly comfortable and enjoyable!

MAMALU PRO™ combines functionality and comfort, providing users with unforgettable experiences during every outing. The adjustable reclining angle of the backrest allows you to customize the stroller to your child's preferences and needs, ensuring perfect support and comfort.

The Comfort upholstery used to finish the MAMALU Pro™ stroller guarantees comfort for the user. This upholstery not only offers aesthetics but also meets the highest standards of safety and hygiene.

MAMALU PRO™ gives you full control over the experience of strolling with your child. The backrest can be easily adjusted, allowing the user to find the most comfortable position for resting or observing the surrounding world. It's more than just a stroller – it's your companion for exploring the world together with your child.

Remember to read this user manual carefully. You will find not only useful information but also guidelines for safe and effective use of the product.

If you have any questions or remarks, do not hesitate to contact with us:

Medical adviser: mobile +48 516 183 883

Akces-MED Ltd.: ph. +48 17 864 04 77

e-mail: export@akces-med.com

Thank you for purchasing our product!

1. Purpose of user manual

ATTENTION

Before using the product for the first time, the user and caregiver are obliged to familiarize with the user manual and comply with its contents.

Remember that complying with guidelines is highly important. It assures safety and extends durability as well as aesthetics of the product.

The instruction manual contains basic information necessary to properly prepare the product before use, use the product, maintain and clean the product.

This user manual should be kept in an easily accessible place. The manual is intended for people who look after disabled users, as well as for doctors and physiotherapists handling the product.

2. Indications and contraindications for use

These medical devices are intended for people with orthopedic, neurological and cardiological diseases and for people after surgery, with musculoskeletal dysfunctions, who have not acquired the ability to sit independently or who have difficulties in adopting and maintaining the sitting position.

Both for flaccid people with reduced muscle tone and for people with spasticity. The device can also be really useful for people who require long-term relief of one or both lower limbs. It is an irreplaceable aid in the rehabilitation process, both in terms of movement and social integration. The device is great help during everyday activities such as rehabilitation but also during playing, studying or having meals. The selection of the appropriate size of the device depends on medical indications and anatomical parameters, regardless of the age of the user.

2. Indications and contraindications for use

The product is recommended for people in the following cases:

- cerebral palsy – various types and forms,
- brain and spinal cord injuries with lower limbs paralysis and weakness, balance and motor coordination disorders with different etiology,
- demyelinating disease (e.g. multiple sclerosis - SM),
- meningocele,
- muscles diseases with weakness and paralysis (dystrophy, myasthenia gravis, myositis),
- genetic disorders,
- degeneration and neurological diseases,
- other condition connected with muscles weakness, paralysis and musculoskeletal disorders,
- after head and spinal cord injuries and spine trauma with paralysis and paresis.

Contraindications:

- critical condition of the user,
- paralysis,
- deep sedation,
- unstable fractures with specific orthopedic recommendations, including spine fractures,
- untreated deep vein thrombosis,
- hemodynamic instability,
- identified brain dysfunction when sitting upright,
- severe arthritis of the cervical spine,
- pulmonary embolism,
- recent myocardial infarction in the last 24 hours,
- fresh postoperative wounds or non-healing wounds in the back area, pelvic girdle area, hip joints and thighs,
- users with mental disorders should not be left in the device without the supervision of a caregiver.

3. General characteristic of the product

Mamalu pro™

STANDARD EQUIPMENT



FOLDABLE FRAME WITH SEAT DEPTH
ADJUSTMENT

PUSH HANDLES

ADJUSTABLE BACKREST

FRONT SWIVEL WHEELS WITH DIRECTION
BLOCKADE

REAR WHEELS WITH INDEPENDENT BRAKES

5-POINT SAFETY BELTS

ADJUSTABLE FOOTREST





CARRYING HANDLE FOR A FOLDED STROLLER

COMFORT™ UPHOLSTERY COVERING THE SIDES

i INFORMATION

Visit en.akces-med.com to see the available accessories.

4. Size table

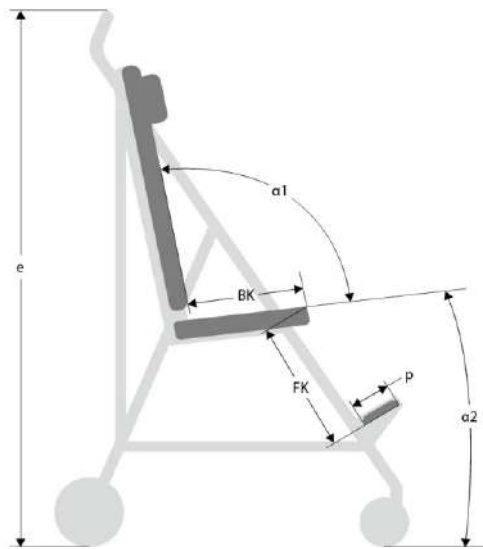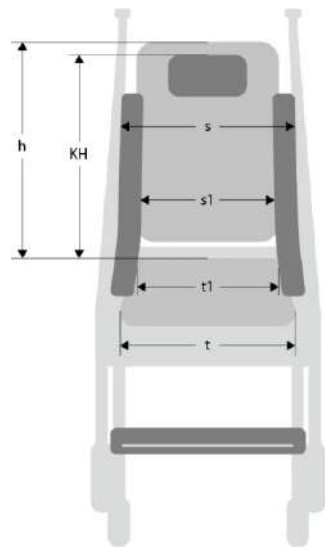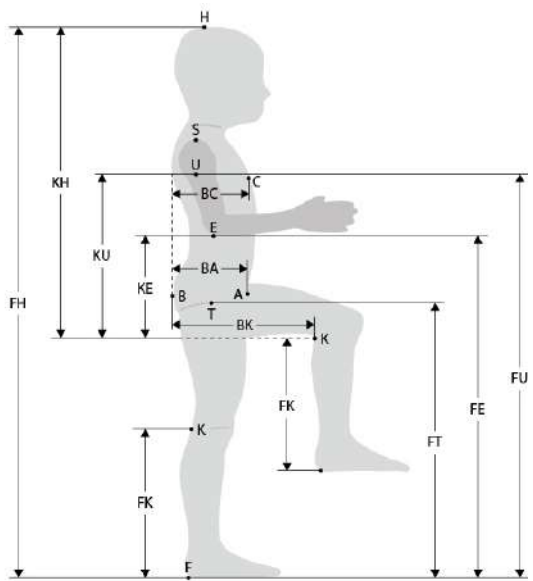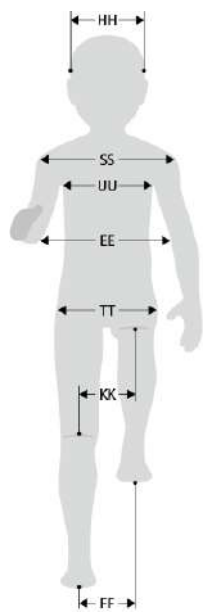

4. Size table

Symbol	Measurement	Unit	Size 1
FH	indicative user height *	[cm]	90 - 130
t	seat width	[cm]	37
t1	seat width with height lateral supports	[cm]	30
BK	seat depth	[cm]	26, 33
s	backrest width	[cm]	37
s1	backrest width with height lateral supports	[cm]	30
h	backrest height	[cm]	58
KH	headrest height	[cm]	43 - 58
FK	distance between footrest and seat	[cm]	12,21, 29, 40
p	footplate length	[cm]	15
e	push handles height	[cm]	93 - 112
α1	backrest tilt angle (relative to the seat)	[°]	90 - 105
α2	seat tilt angle	[°]	23
	max. user weight	[kg]	50

Weight and dimensions	Unit	Size 1
width	[cm]	61
width after folding	[cm]	32
length	[cm]	93 - 113
length ater folding	[cm]	122
height	[cm]	105 - 112
height after folding	[cm]	29
weight	[kg]	13

* Height cannot be the main parameter while choosing the product size.

Note: Approximate values, depending on the product configuration. Measurement tolerance range is +/-1 cm/ ½ in.

5. How to use the product

5.1 Unfolding the product



ATTENTION

Before using the stroller make sure that the front, back and down strut are completely unfolded.

When applying force, spread the struts by pushing them downwards.

5. How to use the product

5.1 Unfolding the product



⚠ ATTENTION

In addition, there are additional safeguards on the sides of the stroller frame to prevent the stroller from unintentionally unfolding.

Before using the product, always check that it is correctly unfolded and that the struts and safeguards preventing accidental unfolding are locked in place. If not locked correctly, the product may fold under the user's weight, which may result in injury.

5. How to use the product

5.2 Folding the product



5. How to use the product

5.2 Folding the product



5. How to use the product

5.3 Backrest tilt-angle adjustment



ATTENTION

Before using the product, make sure that the backrest tilt adjustment knobs are tightened securely and that the backrest does not change position when force is applied.

Improper locking of the backrest may result in a rapid change in the user's position, which may lead to injury and loss of product stability.

5. How to use the product

5.4 Seat depth adjustment



ATTENTION

When adjusting, make sure that there are no upholstery elements between the adjustment elements.

There is a risk of pinching the user's fingers or skin, which may result in skin damage or body injury - adjustments should be made by a caregiver.

5. How to use the product

5.5 Footrest height adjustment



INFORMATION

Press the levers located under the footrest and pull it up firmly.

Lock the footrest at the desired height.




5.6 5-point safety belts



5. How to use the product

5.6 5-point safety belts

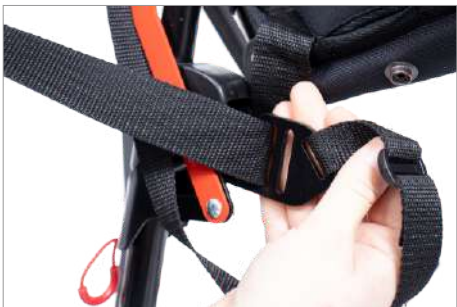

5. How to use the product

5.6 5-point safety belts



INFORMATION

If a vest is used instead of belts, be sure to secure the user's pelvis to prevent slipping, and refer to the information in section 6.6 How to use vests.


ATTENTION

Before using the product, make sure that the 5-point safety belts are correctly fastened. You can pull on the fastened belts to check that it does not come undone when force is applied.

If fastened incorrectly, the user may fall out of the product and suffer permanent injury.

5. How to use the product

5.7 Rear wheels with independent brakes

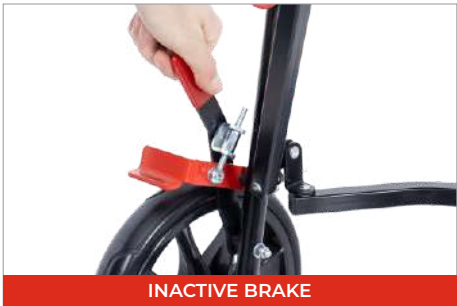


INFORMATION


To activate the brakes correctly, after pushing each lever down, apply additional force and overcome the brake breakpoint. They will lower their level and effectively lock each of the rear wheels.

The brakes have adjustable braking force.

Adjustments can be made by tightening or loosening the nut located on each brake.

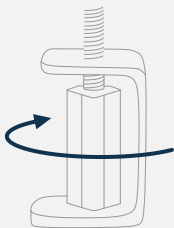



TIGHTEN CLOCKWISE - INCREASE PRESSURE

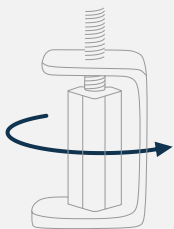

UNSCREW COUNTERCLOCKWISE - REDUCE PRESSURE

5. How to use the product

5.8 Front swivel wheels with direction blockade

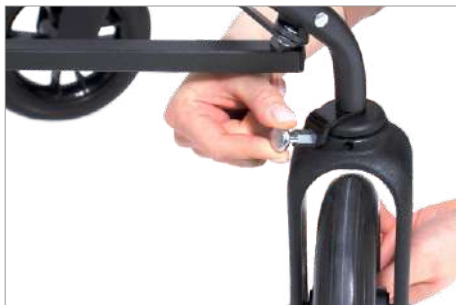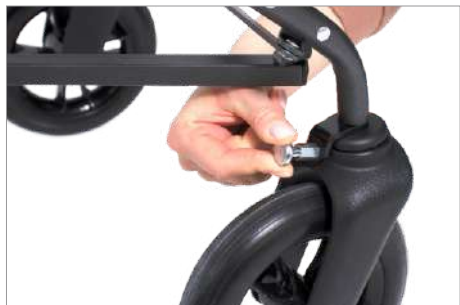

i INFORMATION

Each of the front wheels has the ability to lock the rotation.

Active lock (locking pins inserted into locking grooves) - locks the direction of movement.

Lock inactive (slide out the lock pins and turn them to prevent them from falling into the locking grooves) - the front wheels can be turned in any direction.

5.9 Push handles



6. Use of the stroller

6.1 Preparation of the user

Before using the product, it is essential to make sure that:

- user's outfit is light and comfortable sporty clothes are advisable,
- shoes should be comfortable and, moreover, they should stabilize ankle,
- due to hygienic reason and to avoid abrasion it is advisable for the patient to always wear clothes when using the device, skin must not be naked especially in places of regular contact with upholstery,
- drains, bags, catheters etc. if applied, have to be secured to function properly and not to disturb during using the product,
- if it is necessary for a user to wear orthopedic supply (collar, corset, orthosis, shoes) if it is recommended by doctor.

ATTENTION

Outfit should protect user's skin from direct contact with upholstery as well as other parts of equipment.

Only doctor can make a decision concerning using any kind of orthopedic supply.

6.2 Getting on and off the product

Before use it is necessary to:

- place the product on a even ground without any slope, allowing easy access to the product,
- make sure that all parts are serviceable, correctly mounted and stable,
- block brakes,
- adjust the device to user's measurements,
- undo 5-point safety belts or vest, open safety rail and other accessories, to place easily the user in the device,
- remove the footrest to avoid standing on the footplate,
- place the user in the device (Fig.1),
- fasten the elements used to stabilize the user,

6. Use of the stroller

6.2 Getting on and off the product

- place the support and positioning elements of the user,
- install the remaining equipment and unfold the footrest.

Removing the user from the device:

- place the device on even ground, allowing easy access to the product,
- block brakes,
- undo 5-point safety belts or vest, open safety rail and other accessories, to place easily the user in the device,
- remove the footrest to avoid standing on the footplate,
- pull the user out of the product (Fig.1).

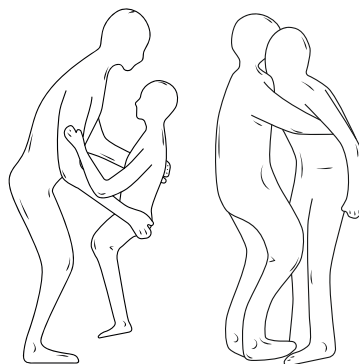


Fig. 1 Ways to sustain the user

⚠ ATTENTION

Before the product is used, it is highly necessary to make sure that all belts as well as safety equipment are correctly fastened as well as adjusted. What is more, check if they do not become undone due to the pressure and, moreover, make sure that clampings from adjustable elements are firmly tightened.

It is forbidden to stand on the footplate when putting the user into the product. In extreme situations, this may result in damage to the footrest or the product tipping over and becoming damaged, which may result in injury to the user.

Equipment works correctly only when the device is adjusted according to user's height as well as weight.

Remember to take all these measurements into account while purchasing the product. Correct adjustment is the most important due to the fact that only then the product is safe. It is forbidden to force child to keep specific position because he/she will feel uncomfortable and it may lead to wrong posture.

In case of users who have problem with bladder, it is necessary to put special cloth on seat. This cloth can be purchased in pharmacy or medical shops.

Improper use of the product can lead to the perpetuation of abnormal posture patterns.

6. Use of the stroller

6.3 Sitting position

The first use of the device usually takes place at a rehabilitation, neurology, neurosurgery or orthopedics center. It is especially important to correctly adjust each element of the device to the user regarding their condition and taking into account existing contractures and deformations.

The first use of the device at home should be done with the help of qualified staff (physiotherapist, health nurse, properly prepared family members) after having read the instruction manual.

ATTENTION

Incorrect device adjustment can lead to such side effects as fainting, body deformities, joint overload. If the abovementioned symptoms occur, stop using the device immediately, take the user out of the device and lay them down. If the undesirable symptoms persist, contact a physician immediately. If the symptoms have disappeared, inform the attending physician about this fact in order to make a decision about the further rehabilitation process.

The decision to use the special needs stroller must always be made by the attending physician.

Adjusting the device to the user should only be done by people who have been properly trained and have read the instruction manual.

6.4 Position when using a special needs stroller

To ensure user safety and health, the pelvis and back should be positioned as close to the backrest as possible to evenly support the spine. The thighs should be supported along their entire length, which requires adjusting the seat depth with the available settings, or if such settings are not available, with suitable accessories.

The feet should rest on the footrest plate to prevent slipping, ensure optimal muscle tension, and transfer body weight.

Due to the available product adjustments, it is prohibited to set a position where the user's head is below the pelvis line. This position poses a risk of choking or aspiration.

6. Use of the product

6.5 Driving a special needs stroller

While using the stroller, always hold it by the handle to prevent it from rolling uncontrollably.

To start moving, release the central brake and, holding the handle, guide the stroller in a safe direction.

Moving on sloped surfaces

When going uphill, always position the stroller facing the direction of travel. Limit your speed and move slowly. The caregiver must have enough strength to control the stroller's stability.

Remember that the center of gravity of the stroller changes on uneven surfaces. Ensure the ramp is suitable and the wheels are high enough so that the frame does not touch the ramp. It is advisable to have the assistance of a second person.

Overcoming thresholds or curbs

To overcome small obstacles, press the handle down to lift the front wheels of the stroller, then roll them over the obstacle. Pull the handle up and lift the rear wheels to move them over the obstacle.

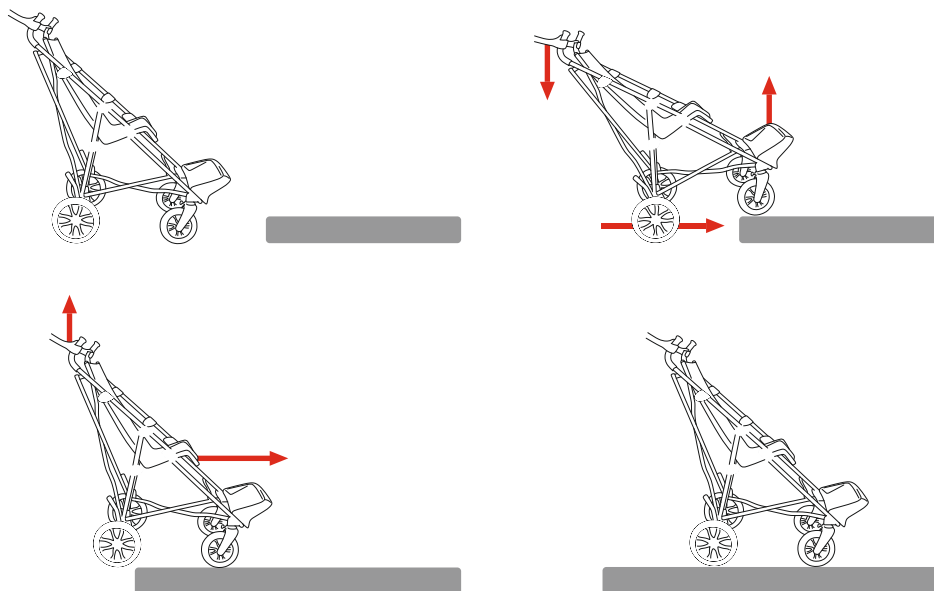


Fig. 2 Diagram showing how to overcome thresholds and curbs

6. Use of the product

6.6 How to use vests

ATTENTION

The vest provides posture support for special needs stroller users as well as users sitting in positioning chairs or other rehabilitation seats. When properly used, the vest allows the user to adopt the correct sitting position for a longer period of time. The design and shape of the vest make it possible for the user to be free to move their upper limbs within the constraints of the user.

If the vest is not used properly, it can pose a serious threat to the health and life of the user.

Possible threats incident to the use of the vest.

1. If the lower vest straps are fastened too loosely, there is a risk that the vest will move upwards to the user's neck, making it difficult to breathe - always check the tension of the straps and set them in such a way that the lower edge of the vest is located near the lower abdomen of the user.
2. In a situation when the user slips off the seat, the upper edge of the vest covers the user's neck and blocks the trachea, making it difficult to breathe - it is therefore necessary to ALWAYS use the accessories stabilizing the user's pelvis, such as a pelvic belt.
3. There is a risk that the upper straps of the vest will press the user's neck, blocking the trachea and making it difficult to breathe. This might happen if the user's body is improperly secured in the device - it is then necessary to use the optional accessories of these devices and stabilize the user's body using the lateral supports. If there is no possibility to use such accessories with the device and the user's body cannot be properly stabilized, the device should be replaced with the one having such an option.
4. The vest is fastened too tightly to the backrest and as a result the user cannot breathe easily – in this case the vest should be loosened in such a way that it is possible for the caregiver to insert a straightened hand between the vest and the user's body. If, however, we want to stabilize the user's body properly, we should use a dynamic vest, which does not cause the breathing difficulty for the user.

Instructions for the proper use of the vest:

1. Make sure that the user is sitting in the best possible position:
 - in the middle of the seat,
 - deeply in the seat in such a way that the user's lumbar spine rests against the backrest.
2. Secure the user's pelvis.

6. Use of the product

6.6 How to use vests

3. Make sure that the side supports of the device, if used, are set in the correct position. It is required that the users with the poor trunk control use the side supports. The side supports limit the lateral movements of the user's body, which could lead to the compression of the vest straps on the neck of the user.
4. Secure the vest:
 - fasten and adjust the lower straps of the vest, so that the lower edge of the vest is located near the lower abdomen of the user,
 - fasten the horizontal strap of the vest (if present),
 - fasten the shoulder straps of the vest,
 - check if the buckles of the vest do not open under pressure.
5. Check the user's position:
 - make sure that the vest cannot move by itself towards the user's neck,
 - make sure that you can insert a straightened hand between the vest and the user's body.

To prevent injuries and to minimize the risks associated with the use of the vest, the following rules should be followed.

1. The position of the vest **MUST** be checked each time after putting the user in the seat.
2. Special attention should be paid to protecting the user from falling forward, slipping off the seat or lifting the pelvis. The vest **MUST** be used with the accessories which will secure the user's pelvis at all times (e.g. the pelvic belt, thigh abduction belts). The use of the vest cannot be considered separately but as a part of a general posture support system including supports such as pelvic belts, foot support, torso stabilization, thigh abduction belts, tilt-in-space position and reclining backrest.
3. Special attention should be paid to ensuring that the entire foot of the user has stable support. If the device is not equipped with a foot support, the user's feet must rest firmly against the ground.
4. It should be remembered that any change of the inclination angle of the backrest, seat, foot support or tilt-in-space function might affect the effectiveness of the belts and supports!
5. Special attention should be paid to the use of stabilization elements or side supports. If the user has poor trunk control, the side supports will limit the lateral movements, which could lead to the compression of the vest straps on the neck of the user.
6. The mounting of the vest and adjusting it to the user can only be carried out by a suitably trained person, who is familiar with the instructions for the use of the vest and the rules of its safe use.
7. Do not make any repairs or modifications to the vest on your own. In order to repair or replace any part of the vest, please contact the manufacturer.

6. Use of the product

6.6 How to use vests

8. Do not use a damaged or incomplete vest, as this poses a serious threat to the health and life of the user!
9. Do not leave the user in the device unattended!
10. The correctness of the use of the vest should be particularly controlled in accordance with the recommendations in case of children, users with impaired memory or mentally ill.
11. Make sure that all people taking care of the user are familiar with the rules of the safe use of the vest.

The vest works effectively only if kept clean and maintained in accordance with the manufacturer's instructions. Before each use of the vest, check whether:

- the vest straps are not frayed or worn. If so, have them exchanged for new ones,
- the vest straps are not attached to the wheels of the device,
- the buckles are not damaged. If so, have them exchanged for new ones,
- the Velcro fastening holds firmly. If not, remove any dirt or hair from the velcro, or have it replaced with a new one,
- there have been no changes in the user's height or weight from the moment the vest was adjusted to the user,
- the shoulder straps of the vest are adjusted so that they are not too loose or too tight,
- the vest should be cleaned regularly with the use of warm soapy water and dried out of the direct sunlight and other sources of heat. The vest must always be dry before usage.

If you have any doubts or questions about the use of the vest, please contact your doctor or physiotherapist.

7. Safety rules

ATTENTION

The special needs stroller MAMALU PRO™ has to be used with compliance to guidelines included in this user manual.

Do not forget to prepare both product as well as user before using the device.

It is highly important to consult either doctor or physiotherapist in order to make right decision concerning the choice of stroller and its size. It is also essential to familiarize with the guidelines described in user manual before using the device.

1. Before use it is necessary to:
 - make sure that there are no damages or different faults which could endanger user's life,
 - check if all parts are working and are correctly mounted and, moreover, check if they are not broken,
 - check if all screws, knobs and all glued, sewed or tapped parts are tightened,
 - check the condition of the upholstery for signs of damage,
 - check if brakes are operating correctly,
 - check if buckles are operating correctly,
 - always use safety belts, Velcro tapes should overlap min. 8 cm,
 - adjust the device according to user's measurements and needs,
 - make sure that all movable parts and knobs for adjustments are stable and locked,
 - secure user with the aid of elements such as vest, feet stabilizing belts, supports, abduction block, headrest. It is highly important to secure user because it safeguards him/her from falling out,
 - if abduction contracture occurs, it is recommended to use abduction block in order to prevent ulcers between knees.
2. It is forbidden to use stroller for different purposes that it is intended to.
3. It is forbidden to use stroller when it is damaged or any part is missing.
4. It is forbidden to leave the user unattended.
5. Device should be used only when there is someone who is trained and knowledgeable.
6. It is forbidden to leave stroller and the user on unstable ground unattended, even if the brake is engaged.
7. It is forbidden to either go up or down the stairs; it doesn't matter if the user is placed in the device or not.
8. The user should always get on and off the product with the brake engaged.

7. Safety rules

9. It's important to make sure that user doesn't stand on footer during either getting into or getting out the stroller.
10. Make sure that all limbs are not close to spokes or other adjustable elements.
11. Please pay your attention when approaching to sources of fire or sources of high temperature during using the device. Moreover, stroller shouldn't be stored in rooms with high humidity, basements, outdoors - balconies or terraces. If it is necessary and the product will be stored for a longer period of time outside rooms with low humidity, it should be protected with a waterproof cover.
12. Secure device during transportation in order to avoid any possible damages. Manufacturer doesn't take responsibility for damages which appeared due to improper preparation of the device during transportation.
13. Before using the stroller, remove all paper and plastic packaging. The packaging is not a toy! Keep away from children and animals.

ATTENTION

Incorrect product adjustment can lead to such side effects as fainting, body deformities, joint overload, etc. If the abovementioned symptoms occur, stop using the product immediately, take the user out of the product and lay them down. If the undesirable symptoms persist, contact a physician immediately. If the symptoms have disappeared, inform the attending physician about this fact in order to make a decision about the further rehabilitation process.

Serious incidents involving this device should be reported both to manufacturer and to the competent authority in your country.

It is forbidden to place the user in the device while being in car, bus or train.

The product contains small parts which children may swallow.

Do not use a product that has any signs of damage, tearing or wear. Only an undamaged product and proper use guarantee its right and safe working. If the product is damaged, discontinue use and contact the manufacturer.

Under no circumstances should an unqualified person repair or alter the product and its construction, as the health and safety of the user depend on it as well as it will lead to the loss of warranty.

The weight of user should not be bigger than maximal weight described on page 8 where you will find information regarding this issue.



8. Terms of warranty

1. Akces-MED Ltd. sells products free from manufacturing defects or faults.
2. Manufacturer gives the purchaser 24-month guarantee from the date of purchase. All defects revealed during warranty period will be repaired free of charge.
3. All defects which appear after warranty period can be repaired only when the user will pay for it.
4. Manufacturer claims that the product can be used for 48-months. User can use this product after this period only when he/she will send the device to manufacturer so as to service it. Next, manufacturer determines again how long the product can be used and, moreover, the date of the next service will be determined.
5. If product is damaged, please contact our dealer or the Akces-MED Ltd. Customer Service Department (ph. +4817 864 04 77, e-mail: export@akces-med.com).

9. Maintenance and cleaning

The special needs stroller MAMALU PRO™ which is a medical device, should always be kept clean and should be used in accordance with the manufacturer's instructions.

The stroller should be kept in a clean, dry and ventilated room.

Maintenance and cleaning must be done at least once a month. Periodic cleaning and maintenance of the stroller is essential to keep its proper functioning and safety.

Recommendations for cleaning the upholstery and other elements of the product.

1. Wash with warm water using a soft cloth and mild cleaners.
2. Make sure the cloth is not too wet, as this may enlarge the stain and damage the upholstery or foam inside the upholstery.
3. After cleaning the stains, the upholstery should be rinsed with lukewarm water and dried with a clean cloth.
4. Do not allow the upholstery to get soaked during cleaning.
5. The upholstery should not be soaked.
6. Remember to remove excess moisture from the upholstery before the next cleaning.
7. Do not use hard brushes for cleaning the upholstery.
8. Do not clean the upholstery with strong chemicals, e.g. solvents or chlorine-based caustics.
9. Do not clean, replace any elements or maintain the stroller during use.
10. Metal and plastic elements should be cleaned by hand with suitable cleaners, after having read the recommendations of the manufacturer of the given cleaner.
11. It is possible to disinfect metal, plastic, wooden and upholstery elements of the product with a generally available disinfectant. In order to disinfect the selected element, spray its surface with a disinfectant generally available on the market and wipe with a dry, clean cloth. After disinfection, wait until the element is completely dry, especially in the case of upholstery. Before using the disinfectant, first check in an invisible place whether the product will not discolor or damage the disinfected surface.



9. Maintenance and cleaning

Before using it is necessary to:

- clean wheels,
- check if base as well as wheels are correctly mounted, check if brake stops turning wheel,
- check if product construction is free of breaks or any kinds of deformities,
- check if all movable parts are working and all screws, nuts, knobs etc. are correctly tightened,
- check if belts are not frayed, check if buckles and clamps work correctly.

ATTENTION

Remember that upholstery has to be always dry before using!

The manufacturer of the special needs stroller, Akces-MED Ltd., is not responsible for any damages caused by using unsuitable cleaners.

The manufacturer is only liable for hidden defects or technical damage caused during normal use of the product. Only if these recommendations are strictly followed is the manufacturer liable for the proper functioning of the product.

In the event of any damage, please contact the authorized dealer from whom you purchased the product or the Customer Service Department of Akces-MED Ltd. (ph. +4817 864 04 77).

9. Maintenance and cleaning

TECHNICAL INSPECTION

Technical inspections should be performed regularly. Before servicing, the product should be thoroughly cleaned to reveal any hidden damage.

BEFORE EACH USE

GENERAL CONDITION OF THE PRODUCT

Check:

- the entire product for any visible damage, cracks or mechanical defects,
- if the product is complete,
- if all bolts, screws, knobs, nuts and other elements of the construction are complete and tightened.

WHEELS AND BRAKES

Check:

- if the wheels are properly mounted and if they are not damaged,
- if the brake is blocking the spinning wheel,
- if the wheels are not dirty. If necessary, clean the wheels.

FASTENING OF THE CONSTRUCTION ELEMENTS

Check all the mechanisms of the product and its moving parts. Check in particular:

- if the seat is properly mounted (if applicable),
- if the wheels are properly mounted (if applicable),
- if the elements securing the user are properly mounted and if they work well (if applicable),
- if all the moving parts are complete, work properly, do not jam,
- if the adjustment mechanisms of the following elements work properly (if applicable): headrest, backrest, seat, abduction block, armrests, footrest.

UPHOLSTERY ELEMENTS

Check:

- if there is any damage, tears, rips, loose seams or deformations on the upholstery elements,
- if the stabilizing straps, vests or safety belts are not frayed or twisted and if the buckles work properly,
- if the stabilizing straps, vests or safety belts work correctly and do not unfasten under pressure or when pulled.

In case of any damage or defects, discontinue the use of the product and contact your Supplier/Distributor or the Manufacturer.

ONCE A WEEK

PRODUCT CLEANING

If the frame and upholstery get dirty (dusty, muddy, etc.) during everyday use, wipe them with a damp cloth without detergents. Remove dirt (e.g. hair, food remains, etc.) from the moving parts of the product.

The product should be cleaned when necessary, but at least once a week. Do not use any chlorine based cleaners or methylated spirit. Before using the product, check if the upholstery is dry.

9. Maintenance and cleaning

ONCE A MONTH

WHEEL AXLES, QUICK CONNECTORS (IF APPLICABLE)

Check:

- if the wheels are rolling and if they are not loose. To maintain the wheels, apply a small amount of a preservative lubricant between the axle and the wheel hub. Check if you can easily remove and attach the wheels.

If dirty, clean the wheels from food residue, dust, hair or other.

PRODUCT CONSTRUCTION CONNECTORS (WELDING, THREADING, SOLDERING)

Check:

- the condition of the product frame in the places of joining (welds, threads, soldering).

TIRES/INNERTUBES (IF APPLICABLE)

Check:

- tire wear condition. Check for visible damage. Check tire pressure (if applicable).

If the tire pressure is too low, inflate them to the correct level.

In case of any damage or defects, discontinue the use of the product and contact your Supplier/Distributor or the Manufacturer.

ONCE EVERY 6 MONTHS

CONSTRUCTION MOVING PARTS

Check:

- if the moving parts of the construction have not been damaged. Maintain them using a proper cleaning agent.

PRODUCT LABELLING

Check:

- if the marking and labels are readable.

In case of any damage or defects, discontinue the use of the product and contact your Supplier/Distributor or the Manufacturer.

AFTER THE LIFETIME OF THE PRODUCT FINISHES

DETAILED PRODUCT CHECK

The manufacturer specifies the life time of the product, which is given in the user manual. After the given period, the user may use the product only if it is sent back to the manufacturer to be checked first. After that, the manufacturer will re-define the life time of the product and the date of the next check.

If you have any questions, please contact Supplier/Distributor or the Manufacturer.

10. Labelling



Declaration that the product meets the essential requirements



Serial number



Please read the manual instructions before use



Attention



Manufacturer



Production date



The product does not contain latex



Store in a dry place



Protect from sunlight



Storage temperature limitation



Medical device



Unique Device Identification



Dry vertically



Do not chlorinate



Do not iron



Do not dry clean



Do not bleach



Careful! Delicate content



It is forbidden to either go up or down the stairs. It doesn't matter if the user is placed in the device or not



Prohibition to leave the user without care during use of the product



Prohibition of use without fastened safety belts



You need to read the additional information



You need to press the button shown in the photo



You need to use the tools indicated in the assembly description

The product data plate contains basic product information such as name, maximum load, manufacturer's data, date of manufacture, and marking in accordance with the requirements for medical devices.



Akces-MED Ltd.

Jasionka 955 B,
36-002 Jasionka, POLAND
ph. +48 17 864 04 77
e-mail: export@akces-med.com
BDO: 000031619

Authorised Dealer

De of issue: 27.12.2023 | The last update: 04.02.2026

© Sławomir Wroński 2026