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EN_English

Thank you for choosing a Fumagalli Care&Reha product.

Instructions for use

WARNINGS

IMPORTANT - READ CAREFULLY AND KEEP FOR FUTURE REFERENCE.

The child's safety could be at risk if these instructions are not followed.

- _ This product was designed for the transport of 1 child in the seat.
- _ Do not use this item for a number of occupants exceeding that envisaged by the manufacturer.
- _ **WARNING** The assembly and preparation of the product must be carried out by an adult. Postural positioning operations and training on the use of the item must be carried out only by authorised orthotists or trained healthcare professionals.
- _ Do not use this product if there are parts missing or signs of breakage.
- _ **WARNING** **Ensure that all the locking devices are engaged before use.**
- _ **WARNING** **Check that the seat unit attachment devices are correctly engaged before use.**
- _ Always apply the brake when stationary and when loading or unloading children.
- _ Do not insert fingers into the mechanisms.
- _ Any weights hanging on the handlebar or handles could make the item unstable; follow the manufacturer's instructions regards the maximum loads transportable.
- _ Do not load the baskets over 10 kg. Do not put weight in the cup holder (if present) in excess of those indicated on the cup holders themselves and never hot drinks. Do not put loads of more than 0.2

kg in the hood pockets (if present).

– **WARNING To avoid injury ensure that the child is kept away when unfolding and folding this product**

– **WARNING Do not let the child play with this product.**

This product is not a toy.

– **WARNING Never leave the child unattended.**

– Be careful not to hurt the child when adjusting the mechanisms (handlebar, backrest ...).

– Do not use the product near to stairs or steps, heat sources, naked flames or dangerous objects within reach of the child.

– Only replacement parts supplied or recommended by the manufacturer/distributor shall be used.

– **WARNING This product is not suitable for running or skating.**

– **WARNING Do not add any mattress not approved by the manufacturer.**

– Do not use accessories which are not approved by the manufacturer.

Chassis + Seat:

This product is suitable for children from 6 months to 6 years or 30 kg in weight, whichever comes first. For babies weighing more than 15 kg, direct and use the seat exclusively following the direction of travel.

– **WARNING: This seat unit is not suitable for children under 6 months.**

– **WARNING: Always use the restraint system.**

Always use the containment belts that also provide postural support.

– **WARNING: When the seat is fitted onto the chassis the presence of little gaps or rocking between the base and the seat do not compromise the safety of the product.**

– The front bar has not been designed to support the child's weight; the front bar has not been designed to lift the seat with the child's weight. The front bar does not replace the seat belt.

– **WARNING:** do not use the rain canopy (if present) in interiors and check the child is not too hot; do not leave near sources of heat and keep away from cigarettes. The rain canopy must not be used on a stroller without a hood or parasol it can attach to. Use the rain canopy only under adult supervision. Check the rain canopy does not interfere with any moving mechanism in the chassis or stroller. Remove the rain canopy before folding the chassis or stroller.

"Intended use: is designed for children with special needs (GMFCS level I to V, up 30 kg in weight), who need a foldable, easy-to-manage stroller for travelling short distances."

Declaration of conformity: This product complies with the requirements of Regulation (EU) 2017/745 on medical devices. If the product is modified, used in combination with another product made by another manufacturer or used with non-original accessories or replacement parts, the CE marking must be removed.

The images referring to the textile details may differ from the collection purchased.

INSTRUCTIONS

CHASSIS

1 OPENING: Before opening the frame, if present, remove (without using blades) all die-cut cardboard.

WARNING:

– DO NOT PULL OR FORCE THE FASTENER BEYOND ITS LIMIT.

– DO NOT USE THE FASTENER AS A HANDLE BAR FOR TRANSPORT OR TO CARRY WEIGHTS.

– DO NOT HANG BAGS OR OTHER ACCESSORIES ON THIS ELEMENT.

Press the side fastener downwards, grip the upper ends of the handle bar and rotate them towards you until the chassis has opened completely (fig_a).

Opened chassis (fig_b). The chassis is opened correctly

- when you hear a click, which corresponds to the handle bar frame attaching - please look at the picture.
- 2 **FITTING THE WHEELS:** slot the rear wheel crosspiece onto the uprights (fig_a), ensuring that it is facing the right way, then attach the rear wheels until they click into place (fig_b) . To assemble the front wheels, fit them into the slots until they lock into place (fig_c).
 - 3 **FITTING THE HANDLE BAR:** insert the handlebar into the frame (fig_a) as shown in the picture. For the attachment to be complete, press the two buttons on the side of the handle bar (fig_b) and press the handle bar until you hear it click (fig_c).
 - 4 **FRONT BASKET ASSEMBLY:**
Slip the basket flap into the slot along the front bar of the chassis (Fig_a) and fasten the basket eyelets to the buttons on each side of the frame (fig_b) .
To complete the basket attachment procedure for the front part, you are required to insert the supplied tie (fig_1) as follows:
position it between the basket and buttons, as shown (fig_2) and use your finger to push downwards, so as to complete the attachment procedure (fig_3).
Repeat the same procedure for the other side of the basket (Fig_c).
It is possible to lower or raise a part of the basket for more space (Fig_d).
 - 5 **REAR BASKET ASSEMBLY:**
pass the basket excluding the two tubes (Fig_a).
fasten the basket buttons located on the rear bar (Fig_b)
fit the Velcro including the closest tube (Fig_c)
button and close the Velcro laterally on both sides (Fig_d)
It is possible to open the basket to have more space (Fig_e).
 - 6 **FIXED OR SWIVELING FRONT WHEELS:** Lift the levers to prevent the wheels from swiveling (fig_a) and lower them to allow the wheels to swivel (fig_b). On uneven surfaces, the wheels should be set to their fixed position.
 - 7 **BRAKE:** press the lever down to apply the brake and lock the frame. Do the opposite to release the brake.
 - 8 **ADJUSTABLE HANDLE BAR:** to adjust the height of the handlebar, pull the levers towards you (fig_a) and move the handle bar until it clicks (fig_b). The handle bar can be adjusted in several positions.
 - 9 **FOLDING:** put the handle bar to its lowest position (fig_a). Rotate the safety lever to the left (fig_b) and pull the frame's internal handle towards you (fig_c). Lift the handle until the frame is completely folded (fig_d).
 - 10 To fold correctly, make sure the side hook is engaged (fig_a). When folded, the chassis will stand upright (fig_b).
TRANSPORT: the stroller can be conveniently carried (with the front wheels in fixed position) using the central handle bar (fig_c).
 - 11 All the wheels can be removed to reduce dimensions further. To remove the back wheels: pull the lever downwards and remove the wheel as illustrated (fig_a). To remove the front wheels: press the tab as shown and remove the wheels (fig_b).
- Open the seat from the ends, pulling until it clicks.
- 2 **ATTACHING THE SEAT ONTO THE CHASSIS:** To fasten the Seat, position it on the chassis (fig_a) and press down with both hands until it clicks (fig_b). Ensure the Seat is properly attached.
 - 3 The Seat can be attached onto the chassis in "world-facing" (fig_a) or "mum-facing" position (fig_b). Always engage the brake on the chassis before fastening and unfastening the seat.
 - 4 **REMOVING THE SEAT FROM THE CHASSIS:** To unfasten the Seat, press the levers near the chassis attachments (fig_a) and lift the seat at the same time (fig_b).
 - 5 **ASSEMBLING THE PADS:** attach the side pads' Velcro straps to the seat's pad.
Turn over the reducers from the front.
Reducers attached correctly.
 - 6 Open the buckles by pressing on the extremities and detach (fig_a). Open the abduction belt as shown to make it easier to position the reducer (fig_b).
 - 7 Position the reducer into the seat leaving the abduction belt on top of it (fig_a).
Slip the Velcro strap through the seat eyelet (fig_b) and complete the attachment process by making the Velcro stick under the seat. Close both buckles (fig_c).
 - 8 Attach both black pads, including the abduction belt between them (fig_a), attach both Velcro straps under the seat pad (fig_b) and fasten the press-stud button (fig_c).
 - 9 Complete by fastening the buttons of the seat's side pads (fig_a) on both sides. Pads installed correctly.
 - 10 **FRONT BAR:** Attach the front bar in the slots as shown (fig_a) until you hear a click.
The front bar can be removed by pressing the two buttons at its extremities (fig_b), or opened on one side by pressing only one of the two buttons (fig_c).
 - 11 **HOOD:** To use the hood, pull it out completely until it clicks into place.
 - 12 The hood comes with a mesh insert so that the child can be seen from any position. The hood can be further expanded by unzipping it and pulling it forward.
 - 13 Slide the hood's attachments on the uprights (fig_a) to change the distance between the hood and the seat (fig_b).
 - 14 **BED POSITION:** the seat can be adjusted in three positions that range from "sitting" to "lying", with an intermediate position (fig_a).
Before adjusting the seat to a flatter position, the handlebar must be extended (fig_b). You can switch from one position to another by pulling the lever shown in the picture (fig_c).
 - 15 **ADJUSTING THE INCLINATION OF THE BACKREST/SEAT:** extract the 2 spring pins (one per side) and rotate them to block them outside their slot (fig_a). When starting from an upright position, pull the levers on the rods (fig_b). Adjust as necessary and proceed in reverse order, re-inserting the pins (fig_c).
 - 16 **ADJUSTING THE DEPTH OF THE SEAT:** by opening the sack's side zip, the seat depth can be increased. When increasing the depth, the seat's pad and the side pads must be replaced with the larger ones (accessory provided, figure 36).
 - 17 **ADJUSTING THE HEADREST:** it can be adjusted in 3 positions. To adjust:
open the zip on the seat's backrest (fig_a),
release both Velcro straps to remove it from the backrest (fig_b),
remove the headrest from the front (fig_c),
reposition it by inserting the Velcro straps at the desired height (fig_d) and fixing them onto the backrest. Close the zip.
 - 18 **USING AND ADJUSTING THE CONTAINMENT BELTS:** fasten the buckles until you hear a click. Adjust their adherence to the child by pulling the strap

SERIAL NUMBERS

- 12 Chassis, has the information on the product's production date on the lower part of the frame.
_ Product name, production date and serial number of the chassis. This information is essential in the event of a claim.

INSTRUCTIONS SEAT

- 1 **OPENING:** Before opening the Seat, remove (without using blades) any die-cut cardboard.

- towards you (arrow 1). To open the buckles, press the extremities as shown (arrow 2) and unfasten.
- 19 **ADJUSTING THE CHEST BELT:** it can be adjusted in 2 positions.
To adjust:
open the zip on the seat's backrest and unfasten the two Velcro straps to remove them from the backrest (fig_a),
remove the pad from the front (fig_b),
reposition it by inserting the Velcro straps at the desired height (fig_c) and fixing them onto the backrest. Close the zip.
The pad will adhere more or less closely to the child depending on how you position the Velcro fasteners.
- 20 **ADJUSTING THE DEPTH OF THE FOOTREST:**
To adjust the depth of the footrest, loosen the two knobs under the seat (fig_a) and push the two tubes forwards or internally (fig_b).
- 21 To pull the footrest out, push the two tubes outwards as far as they will go in the direction of the footrest (fig_c).
To pull the footrest back, do the opposite, pushing the tubes inwards as far as they will go (fig_d).
When you have chosen the position you require, tighten the two knobs.
- 22 **ADJUSTING THE HEIGHT OF THE FOOTREST:** the height can be adjusted as needed. Open the two zips at the front (fig_a). At the back of the footrest, unscrew the knobs that loosen the footrest (fig_b). Reposition it as needed by moving it up or down (fig_c). Once it is at the desired height, screw on the knobs at the back to fix the footrest. **IMPORTANT:** always close the two zips (fig_d).
When raising or lowering the footrest, it is advisable to adjust the height of the seat pad in section 24.
- 23 **FOOT BELTS:** To open the buckles, press the extremities and unfasten (fig_a). There are adjustable buckles underneath for greater adherence to the foot (fig_b). To loosen the foot belt, adjust the buckles (Arrow 1); to tighten the belt around the feet, pull as shown by arrow 2.
- 24 **ADJUSTING THE SEAT PAD:**
there are 3 positions (fig_a).
Before adjusting it into the desired position:
- unbutton the side pads,
- unfasten the Velcro strap underneath the seat of the upper pad,
- remove them and set them aside.
Unfasten under the seat the two Velcro straps of the abduction belt (fig_b)
from the front and extract it upwards (fig_c).
The abduction belt must be extracted and repositioned at the same height chosen for the seat; there are 3 available positions (fig_d).
Now you can reposition it.
- Having repositioned the upper pad and the abduction belt, reposition the other elements at the same desired height:
- position on the front seat the upper pad and fasten its Velcro strap under the seat (section 7 a-b),
- the side pads with Velcro fasteners, under the upper pad and the two side buttons to attach to the seat (section 9),
- attach the abduction belt's Velcro straps under the seat and button them up again from the front (section 8 b-c).
- 25 **FOLDING THE CHASSIS WITH THE SEAT ATTACHED:**
the Seat can be folded while still attached to the chassis only if it is world-facing.
To do so:
- fold down the hood,
- close the footrest by pushing it inwards (fig_a),
- remove the front bar (fig_b),
- extract the 2 spring pins (one per side) and rotate them to block them outside their slot (fig_15 a),
- press the levers on the upright (fig_c) and fold the Seat on itself (fig_d).
- 26 **WARNING: If the seat is mum-facing it cannot be folded attached onto the chassis. It must be unfastened (fig_a).**
It can take up less room when folded (fig_b).
- 27 **OPENING THE CHASSIS WITH SEAT ATTACHED:**
Press the side attachment fastener downwards, grip the upper ends of the handle bar and rotate them towards you until the chassis opens completely (fig_a). Chassis open (fig_b). A "click" sound means it has opened correctly and corresponds to the attachment onto the base. Next, open the Seat from the ends, pulling it until it clicks (fig_c).
- 28 **FOLDING THE CHASSIS WITH THE SEAT FASTENED AND FOLDED:** lower the handle bar to the lowest position (fig_a). Rotate the safety lever to the left (fig_b) and, at the same time, pull the frame's internal handle bar towards you (fig_c).
Lift the handle bar until the frame is completely folded (fig_d).
To fold correctly, check the side fastener is engaged. If it has not engaged automatically, compress the frame slightly. The folded stroller is free-standing.
- REMOVING THE FABRIC CASES:**
- Unfasten the seat from the chassis.
- Unfasten the front bar from the seat.
- 29 To remove the hood cover: unfasten the 2 buttons near the pivots (fig_a), then press the buttons on the two attachments and remove the hood (fig_b). Lastly, extract the two rigid supports (fig_c) and detach the hood.
- 30 Open the zip under the seat that attaches the sack to the horizontal bar (fig_a).
Unfasten the two buttons at the side of the seat, on both sides (fig_b).
Fold the seat slightly by pressing the hooks on the side slightly upwards (fig_c).
- 31 Remove the sack fabric on the upper part (fig_d) by pulling it over the backrest adjusting handle (fig_e).
Open both zips on the footrest,
unfasten the Velcro strap and lift the middle fabric flap between the two zips (fig_f).
- 32 Remove the sack from the tube (fig_g) and extract the elastic (fig_h) removing the whole sack.
- Open the zip on the backrest,
- unfasten all Velcro straps on the backrest and seat,
- lastly, extract the rigid ribbing from the sack (fig_i).
- 33 Remove all the pads that are present by unbuttoning them as shown in the picture and unfasten all Velcro straps as shown (fig_a - b).
Under the seat there is a Velcro strap that must be unfastened so that the seat's pad can be removed from above (fig_c).
- 34 The cover of the headrest's pad can be removed by opening the zip (fig_a).
The black pads inside the chest belts can be removed by unfastening the Velcro straps (fig_b).
- 35 The cover of the two small side pads can also be removed by opening the zip (fig_c).
The cover of the seat pad can be removed by detaching the Velcro strap from under the seat itself and opening the zip (fig_d).
- 36 A larger seat reducer is provided; it must be used when increasing the seat's depth (section 16).
- Postural additions may be made to the seat to better meet the child's needs. There are many additional features/options that an orthotist may use, such as the option of inserting containment wedges in the posterior part of the backrest (lumbar**

support, lateral trunk supports, etc.) using extra Velcro straps or the ones that are already present. It is also possible to add leg containment by attaching two wedges under the leg padding.

WARNING: these additions can only be made by an orthotist or healthcare professional.

SERIAL NUMBERS

- 37 Seat includes the label identifying it as a Fumagalli product. The information relating to the seat's production date can be found under the sack, at the side.
To view it, the sack's two side buttons must be unbuttoned (fig_a).
Seat includes the label identifying it as a product on the chassis. The information relating to the production date can be found under the sack (fig_b).

- The product name, production date and serial number of the chassis.
This information is required for any complaints.

PRODUCT DISPOSAL

To help protect the environment, after using the product be sure to dispose of it in accordance with the instructions provided by your local authorities.

THE ORIGINAL ACCESSORY PEG PEREGO

- 38 The rain canopy is sold separately.
To attach it, unfasten the hood's button and fasten the rain cover's button. The rain cover has a zip that allows you to open it easily.
Accessories are designed as a useful, practical means of simplifying the lives of parents.

CLEANING AND MAINTENANCE

Cleaning must only be carried out by adults.
PRODUCT MAINTENANCE: protect from the weather: water, rain, or snow; prolonged and continuous exposure to bright sunlight may give rise to colour changes of various different materials; store this product in a dry place. Conduct regular checks and maintenance on all the main parts
CLEANING THE CHASSIS: regularly clean plastic parts with a damp cloth; do not use solvents, degreasers or aggressive detergents; keep all metal parts dry to prevent rust; using a damp cloth, keep all moving parts (adjustment mechanisms, securing mechanisms, wheels...) free of dust or sand. Regularly clean the wheel pins with a dry cloth; do not use silicone-based sprays.

FUMAGALLI CARE&REHA

This ISO 9001 certified company has many years' experience in providing aids for disabled people. Please do not hesitate to contact us for further information and/or requests.

FUMAGALLI CARE&REHA ASSISTANCE SERVICE

If any parts of the item are accidentally lost or damaged, only use genuine replacement parts. For any repairs, replacements, information on the products and sales of original spare parts and accessories, contact the FUMAGALLI CARE&REHA Assistance Service and state the serial number of the product, if present.

tel. 0039/031/3356811
e-mail service@fumagallicare.com
website: www.fumagallicare.com

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*In accordance with UNI CEI EN ISO/IEC 17050,
we declare under our exclusive responsibility that the Medical Equipment*

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Foldable stroller suitable for children with special needs (GMFCS level 1-5 up to 30 kg in weight), who need a light and easy to handle stroller for short trips. The seat can be customized to meet non-complex postural needs, it is equipped with a footrest adjustable in height and inclination.

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complies with the provisions of the Regulation (EU) MDR 2017/745 relating to medical equipments and was marketed with the CE marking.

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Harmonized norms and technical norms used:

- *UNI CEI EN ISO 14971 - Application of risk management to medical devices*
- *UNI EN ISO 1888 Articles for childcare, prams and pushchairs - Part 2: pushchair for children over 15 kg up to 22 kg. (some tests have been modified by increasing the mass of the tests)*
- *UNI EN ISO 3758 Textiles - Maintenance labeling code using symbols*
- *UNI EN 12182 Products intended for the assistance of people with disabilities - General requirements and test methods*
- *ISO 9999 Classification Technical aids for disabled people*

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Attachment 1
Configuration and component identification codes

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