Hercules Small

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2. Foreword / Manufacturer

Dear customer,

You have chosen our product and we would like to thank you for the trust you have placed in us. To ensure that the handling of your new product is safe, practical and comfortable for you and the user, please read these instructions for use first. This will help you to avoid damage and operating errors. Please ensure that these instructions for use remain with the product and are available to every user.

If you have any questions, please contact your specialist dealer or us directly.

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Phone: +49 (0) 4761 8860

www.thomashilfen.de

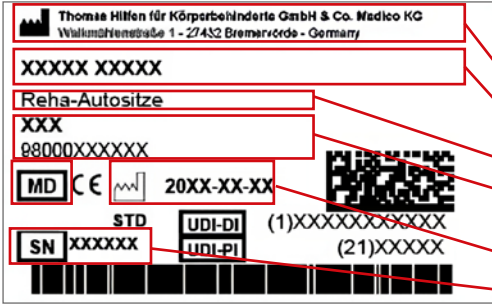
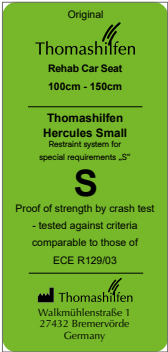
We reserve the right to make technical changes. This may lead to deviations between your product and the representation in media.

Table of contents


1. Figures	2
2. Foreword / Manufacturer	42
3. Labels / model identification.	44
4. Remarks	46
4.1 General remarks	46
4.2 Safety instructions	46
5. Intended Purpose	48
5.1 Intended Use / Application Areas	48
5.2 Indication / Patient Target Group / Intended Users	48
5.3 Risks Associated with Use and Contraindications	48
6. Product description	49
6.1 Scope of delivery / basic equipment	49
6.2 Basic equipment	49
6.3 Variants.	50
6.4 Commissioning.	50
6.5 Use training	50

7. Setting options / performance features	51
8. Accessories.	55
9. Technical data	62
10. Material specifications	63
11. Cleaning, maintenance and repair	63
11.1 Cleaning, care instructions and disinfection in everyday life	63
11.2 Maintenance and inspection by the specialist trade / Maintenance schedule . .	64
11.3 Spare parts	65
11.4 Storage	65
11.5 Disposal.	65
12. EU Declaration of Conformity	66
13. Warranty	66
14. Useful life	66
15. Service life	66
16. Reuse	66

3. Labels / Model Identification

Aufkleber	Meaning
	Barcode label (exchange image) Position on the product (Fig. 1) - Manufacturer - Product name - Product group - Item number short / long - MD = Medical Device - Date of manufacture - Serial number
	Type plate Add picture Position on the product (Fig. 2) - Manufacturer - Product name - Max. user weight / max. load capacity - allowable user size - CE mark - Follow the instructions for use
	Crashtest-Label Position on the product (Fig. 2) Information: Proof of strength by crash test - tested against criteria comparable to those of ECE R129/03.


Gurtführung / Belt guide / Guide de ceinture

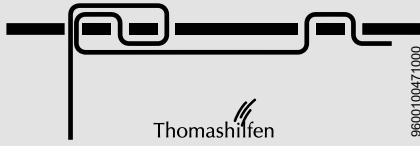**Belt guide 5-point belt**


Position on the product (Fig. 3)

Reference to the belt guide:

- Shoulder straps (back plate above)
- Lap belts (back plate below)
- Crotch strap (seat plate)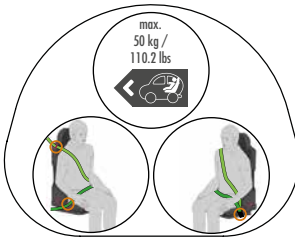

- Driving direction / belt routing
- 3-point safety belt (Fig. 4)

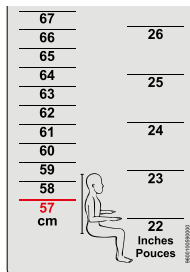- Scale back height
- For easy adjustment of the back height. (Fig. 5)

4. Remarks

4.1 General Remarks

- This "special needs restraint" is designed to give extra support to children who cannot sit properly in conventional seats. To be sure that this restraint system is suitable, the doctor should always be consulted.
- Read the Instructions for Use carefully before first use. If you have difficulty reading, please have them read to you. If the copy of the Instructions for Use is missing, you can download one from www.thomashilfen.de. The contents of the digital Instructions for Use can be enlarged.
- Check the packaging for any transport damage and to be sure the product is in flawless condition.
- Changes to the product are only permitted within the scope of the intended adjustment options.
- Cleaning and daily care can be provided by relatives and caregivers. Maintenance and inspection are done by the distributor.
- All of the screws to the product have to be warmed prior to removal because they are fastened with a thread locker. If the product is to be put back together again, all of the screws have to be secured with a thread locker.
- The back cushion must be pulled over the top of the back plate before first use. This cushion is not pre-assembled!
- In some countries it necessary to carry a medical certificate.
- The rehab car seat can be used on the back seats as well as on the front passenger seat. The chosen car seat **MUST** face forward.
- Only use the seat in an upright position.
- Place the seat in the vehicle so that it cannot be caught by a door or other moving vehicle seats.
- This seat is not suitable for use at home, in boats or other non-certified applications
- All labels have to remain on the product and should not be tampered with.

4.2 Safety Instructions

- Check at least every 3 months that the settings on the product still work for the user's current body measurements. If need be, have a distributor / orthopaedic technician adjust the settings.
- Do regular (weekly) checks to make sure that the setting options on the product work.
- Be careful of the moving parts when adjusting or folding the product.
- Only lift up the product by its firmly welded or screwed parts.

- In some countries (e.g. also in Germany) it is necessary to carry a doctor's prescription (e.g. a copy of the existing prescription for cost coverage of the seat by a cost bearer). This applies in particular if the seat has been subsequently modified from the original Thomashilfen rehab car seat or if accessories such as a swivel base, tilting base or footrest are used.
- Never use the seat's straps to lift or carry the seat.
- Before each use check that all the nuts, bolts, and screws are tight.
- Please do not overload the product and note the maximum user weight and the maximum load capacity (see technical data).
- Never leave the user unattended with the product.
- The product should only be used by one user at a time. No other individuals should be placed in or on the product.
- All users have to be familiar with how the product works. This way in an emergency the user can be removed quickly from the product.
- The user should always be positioned with the five-point positioning harness (standard equipment). The seat and the user always have to be fastened with the vehicle's own three-point seat belt.
- Adjust the straps of the seat so that they fit snugly against the user but do not pinch. Make sure that the straps do not sag or twist.
- Before each ride, check that the seat is securely fastened.
- Even when the seat is not in use, it must be properly secured with the vehicle's own 3-point safety belt. In the event of an accident, an unsecured seat can cause serious injury to the vehicle occupants.
- Strong sunlight and heat can cause high surface temperatures on the product. There is then a risk of injury when touching the product. Do not use the product again until it has cooled down sufficiently. Protect the product from direct sunlight in general.
- Please observe the safety standards applicable to your vehicle in the vehicle's instruction manual to ensure the safety of the user.
- The instructions for use must remain on the product.
- If, contrary to expectations, a serious incident occurs with the product, please report it immediately to our company and the competent authority

5. Intended Purpose

5.1 Intended Use / Application Areas

The Hercules Small rehab car seat is a Class 1 medical device designed for transporting people with disabilities and for maintaining a secure sitting position in motor vehicles for users with a height of 100 - 150 cm and a body weight of 18.1 - 50 kg. The seat is always secured with the vehicle's own 3-point safety belt. The 5-point Belt of the rehab car seat is used to position and support the user in an upright sitting position.

In some countries (e.g. also in Germany) it is necessary to carry a doctor's prescription (e.g. a copy of the existing prescription for cost coverage of the seat by a cost bearer). This applies in particular if the seat has been subsequently modified from the original Thomashilfen rehab car seat or if accessories such as a Tourning plate, Tilting plate or Footrest are used.

The rehab car seat provides additional support for safe sitting, stabilisation of head control and a stable seat in the pelvic area. The seat can be adapted to the growth of the user or the physique of different users by adjusting the Back height, head-rest, pelvic support, Seat depth and leg guidance. The product also offers extensive accessories for customisation. Further required accessories for thorax support can be manufactured by authorised specialist personnel and attached to the available mounting options on the back of the seat.

Any other or additional use is considered improper use.

5.2 Indication / Patient Target Group / Intended Users

The rehab car seat is used for children with structural and / or functional limitations or malpositions in the head and trunk area. It is also used for children who are mentally or neurologically impaired or have perceptual disorders. It provides a stable and physiological positioning when a standard child car seat is too small or not suitable or the vehicle's own safety belt is not sufficient for an upright sitting posture in the vehicle.

5.3 Risks Associated with Use and Contraindications

In order to find the right rehab car seat for the user, the attending physician should always be consulted. With proper use, risks can be excluded during application. Contraindications are not known.

6. Product Description

6.1 Scope of Delivery

Your product is delivered pre-assembled in the basic configuration. Any accessories ordered are enclosed with the product and must be fitted subsequently. Instructions for use are enclosed with the product.

To store your instructions for use, there is a plastic pocket at the back of the product. (Fig. 2)

The instructions for use must always be carried on the product.

6.2 Basic equipment

Rehab car seat consists of:

- Headrest, width and height adjustment infinitely variable. The sides can also be individually adjusted
- Back height and back angle, infinitely adjustable
- Pelvis support, width adjustment infinitely variable. The sides can also be individually adjusted
- Seat depth, infinitely adjustable
- Leg guide, width adjustment infinitely variable. The sides can also be individually adjusted
- Belt guide for the vehicle's 3-point safety belt (can be used on the left and right)
- Mounting fixtures for all accessories
 - o Back cushion,
 - o Seat cushion,
 - o Head cushion
 - o Pelvic cushion,
 - o Leg guide pads,
 - o Seat belt shield pads
 - o 5-point positioning harness including shoulder harness padding
 - o Cushion side impact protection
 - o Cushion joint cover
- Padding set, washable, breathable and flame-retardant

6.3 Variants

Colour variants

- Cushion set grey-red
- Cushion set grey-blue

6.4 Commissioning

- Carefully remove the transport locks and the packaging.
- Make sure that packaging material and small parts cannot get into the hands of children in order to avoid swallowing or the risk of choking.
- The product is delivered folded. Pay attention to the instructions for folding under the item Adjustment options.
- The minimum back height is set when the product is delivered. Before use, ensure that the back height and back cushion are correctly adjusted to the user.

Installation in the vehicle

Hercules Small is placed in the desired position on the car seat. The backrest of the Hercules Small is continuously adjusted to the back angle of the car seat. It is recommended that the back angle is not set below 90°. The 3-point safety belt is first passed through the upper belt guide marked in green (fig. 6), then over the two areas of the pads marked in green in the pelvic area (fig. 7 + 8). Then the belt is locked in the belt buckle with the belt tongue. (Fig. 9) Check the course of the belt. The belt must not be twisted.

Tip: Installation in the vehicle is easier if you unfold the folded Hercules Small on the vehicle seat first.

6.5 User training

The user is instructed in the use of this aid by authorised specialist personnel with reference to these instructions for use.

Car seats only (installation with specialist adviser) Remains in place

The technical advisor should install the seat in the vehicle together with the user and adjust the seat to the respective dimensions of the current user.

It must be ensured that the user has been instructed in all functions of the product as well as the installation and removal from the vehicle.

You can also find detailed instructions for use in videos on the Internet:
www.thomashilfen.de/hercules-small-montage

7. Setting options / performance features

7.1 Wrinkles

Hercules Small can be folded for easy, space-saving transport.

7.1.1 Unfold

Release the red eccentric levers on both sides of the seat to unfold the seat. (Fig. 10) Then secure the seat in the desired position. The seat must be leaning against the car seat.

The pads for the folding joint are attached to the back plate with Velcro for transport and placed behind the back pad. (Fig. 11) Pull out the poster for the folding joint. Position the pads over the folding joint. The green belt marking must be over the indentation in the folding joint. (Fig. 12). Fix the pads to the outside and inside of the sheet metal with Velcro. (Fig. 13)

7.1.2 Fold

First loosen the two pelvic pads on the left and right on the outside (Fig.14) and place them behind the backrest padding. (Fig. 11) This prevents the cushions from being squeezed when folded. Then loosen the two red eccentric levers on both sides of the seat. Now fold the seat and secure it in the folded position.

7.2 Back height adjustment

Determine the user's back height (from the seat to the top of the head).

Loosen the backrest cushion at the top of the backrest plate (Fig. 15). Pull the backrest upholstery forwards. In doing so, the backrest upholstery can be pulled out of the shoulder straps of the 5-point positioning belt (Fig. 16). Now the backrest upholstery can be placed on the seat surface. The screw connection of the back plates is now easily accessible.

Use an open-ended spanner to loosen the four screws until the back plate can be moved. (Fig. 17)

The scale on the back of the back plates helps you to find the right setting. (Fig. 5)

Then fix the four nuts of the back plates at the required back height. (Fig.15)

Adjust the available length of the backrest cushion. To do this, loosen the lower end of the backrest cushion under the seat cushion and pull it out. The remaining length is reattached to the Velcro strap under the seat cushion. (Fig. 18)

Pull the backrest upholstery back over the backrest plate (Fig. 15) Make sure that the shoulder straps of the 5-point positioning belt are again in front of the backrest upholstery and are not twisted. (Fig. 19)

Check that the 3-point seat belt is positioned correctly while driving. The green belt guide with the vehicle's 3-point seat belt must always be above the user's shoulder. (Fig. 20)

7.3 Back angle adjustment

Use the red eccentric levers on both sides of the Hercules Small to adapt the back of the seat to the backrest of the vehicle seat. (Fig. 10)

Make sure that the positioning seat is in full contact with the backrest of the vehicle seat.

7.4 Headrest adjustment

The headrest is used to support and position the head and provides optimal protection for the user. The two sides can be adjusted independently in height and width.

Loosen the backrest upholstery at the top of the backrest plate to gain access to the screw connection of the headrest (Fig. 15).

Loosen the screw connection with an open-ended spanner. Adjust the position of the headrest to the user and fix the screw connection again. (Fig. 21)

Pull the back cushion over the top of the back plate.

7.5 Seat depth adjustment

Determine the required seat depth (backrest to the back of the knee)

The seat depth screw connection is located on the underside of the seat. Place the seat outside the vehicle on the back plate to gain access from below.

to the seat. Lift the seat cushion to gain access to the nuts on the seat. Use an Allen key (underneath the seat) and an open-ended spanner (on the seat plate) to loosen the screw connection of the seat depth adjustment. (Fig. 22 + 23) Move the upper seat plate to the desired depth. Then retighten both screws and position the seat cushion on the seat plate.

7.6 Crotch strap adjustment

Tip: Change the setting outside the vehicle to have enough space.

The crotch strap can be used in two positions on the seat plate. Loosen the belt fastener and remove the harness shield pad and the seat pad from the seat plate. (Fig. 25) Place the Hercules Small preferably on the back plate and loosen the seat depth screw connection. The two screws can be reached from below. (Fig. 22) The nuts can be found on the seat depth plate. (Fig. 23) Pull the seat depth plate out as far as possible. Now you can adjust the position of the crotch strap. Please ensure that it is threaded correctly, refer to the illustration on the seat depth plate.

Crotch strap position for small users (Fig. 25)

Crotch strap position for taller users (Fig. 26)

Now set the desired seat depth again and tighten the two screws again. Thread the harness buckle through the seat pad and then through the harness buckle pad. Finally, thread the lower end of the harness shield 5pad through the seat pad in the same way as the crotch strap. The lower sides of the harness shield pad are now underneath the seat cushion (Fig. 27).

7.7 Pelvic guide setting

The two pelvic guides provide support and positioning in the pelvic area. They can be adjusted independently of each other.

Lift the seat cushion at the back to gain access to the pelvic guide screw connection. (Fig. 28) Loosen the screw connection with an Allen key (from below) and an open-ended spanner (on the seat plate). (Fig. 28 + 29)

Set the desired width and tighten the screws again.

The seat cushion can simply be put back again.

7.8 Leg guide setting

The two leg guides provide support and positioning in the thigh area. They can be adjusted independently of each other.

The screw connection is located underneath the seat plate. If the seat depth is set to the smallest position, the seat depth must first be pulled out in order to gain access to the screw connection of the leg guide. If the seat depth is not set in the smallest position, there is direct access to the screw connection below the seat plate.

Loosen the screw connection with an Allen key and adjust the width as desired. Now tighten the screws again. Check that the seat cushion is correctly seated.

7.9 Adjustment 5-point positioning belt

Please note: The 5-point positioning belt is used to position the user in the seat and does not protect him/her in the event of an accident. The 3-point safety belt provided in the car must always be used to secure the user in the seat.

7.10 Adjusting the shoulder straps - height

Loosen the shoulder straps behind the back pad. (Fig. 30) Then guide them through the strap guides in the back plate at a height suitable for the user. (Fig. 32) Please ensure that the straps are threaded correctly into the back plate, an illustration for this is located directly on the back plate. (Fig. 3)

Note that the shoulder straps must be above the user's shoulder. The upper shoulder belts must be adjusted so that the belt tensioners do not rest on the user's collar-bone. (risk of injury)

7.11 Adjusting the shoulder straps - strap length

Pull the loose webbing straps to shorten and tighten the straps.

To lengthen the belts, press the seat belt pretensioner release button and pull then adjust the belt to the desired length. (Fig. 31) By changing the adjustment range on the belt tensioner, you can adapt the belt even better to the user.

7.12 Adjusting the lap belts - belt length

If the adjustment on the belt tensioners is not sufficient, you can also adjust the lap belt in the lower area of the back plate. To do this, loosen the threaded belt and adjust it to the desired length. Make sure that the end of the belt protrudes at least 5 cm from the slot in the back plate.

7.13 Belt buckle

To open, press the red recess in the fastener. Then separate the two upper strap holders. To close the belt, insert the belt holders one by one into the buckle until they engage with an audible "click".

7.14 Side change belt guide top

Depending on where the Hercules Small is placed in the vehicle, it may be necessary to use the upper green belt guide on the other side of the seat.

Use a hair dryer to loosen the screw lock of the two screws from the black bracket on the back plate (Fig. 33). Mount the bracket on the other side on the back plate with the help of the screw lock and tighten the screws again.

For a correct run of the 3-point safety belt, an adjustment of the belt guide is necessary. Loosen the screw connection of the belt guide on the angle and correct the position of the belt guide, and now tighten the upper screw of the belt guide again. (Fig. 34)

7.15 Upholstery and webbing

Back cushion

The "back" cushion set consists of 4 parts. (Fig. 35)

A = head cushion

B = 2 x head cushion

C = 2 x pads folding joint

D = 2 x pads Side impact protection

Seat cushion

The "seat" upholstery set consists of 4 parts. (Fig. 36)

E = seat cushion

F = 2 x pelvic cushion

G = 2 x leg guide

H = Seat belt shield pad

Seat belt

Set of harnesses (Fig. 37)

J = 5-point positioning belt

K = 2 x shoulder strap pads

8. Accessories

Note: The assembly / disassembly of the accessories is done with standard tools.

- Allen key size 4 + 5 and open-end spanner size 10 + 13 (back plates)
- Allen key size 3 + 4 and open-end spanner size 8 + 10 (seat plates)
- Screw lock (Loctite®)
- Hair dryer (for loosening screwed connections where the screw lock has been used)

You can also find detailed assembly instructions in videos on the Internet:
<http://www.thomashilfen.de/hercules-small-montage>

8.1 Swivel function

With the swivel function, the Hercules Small can be turned to open the vehicle door. This makes it easier to get in and out of the Hercules Small.

To turn the Hercules Small, press the red release button (Fig. 38) and turn the seat towards the open vehicle door. When turning back, make sure that the trigger audibly engages with a "click".

You can also find detailed assembly instructions in videos on the Internet:
<http://www.thomashilfen.de/hercules-small-montage>

The swivel function can also be retrofitted. Changing the direction of rotation (to the right or left) is also possible. However, the assembly is quite complex and must be carried out very carefully by an instructed and qualified person! It is therefore recommended to use the pre-assembly option (right or left can be selected when ordering).

The screw connections are provided with a screw lock. Therefore, the screws must be heated before loosening. Each screw connection must then be provided with a screw lock again (Loctite®).

You can also find detailed assembly instructions in videos on the Internet:
<http://www.thomashilfen.de/hercules-small-montage>

Attention: If the vehicle seat does not have a straight support surface for the positioning seat, the accessory "cushion for function plate" is strongly recommended.

The seat surface of the Hercules Small must rest on as large a surface as possible, especially when using the swivel function.

8.2 Cushion for functional plate

If your vehicle does not provide a straight support surface for the Hercules Small, this can lead to damage to the vehicle seat, especially when using the swivel function. The cushion for functional panels reduces this risk by compensating for the shape of the vehicle seat and thus providing a large-surface support surface for the Hercules Small.

Place the cushion for function panels under the Hercules Small on the vehicle seat. (Fig. 39)

Caution: The use of the cushion may increase the overall height of the Hercules Small. Before driving, check whether there is still sufficient space to the car headliner. A distance of approx. 20 cm between the upper edge of the positioning seat and the headliner is recommended.

8.3 Tilt function

The tilt function facilitates optimal positioning of the user when upper body or head control is limited.

The tilt function allows Hercules Small to be tilted backwards in 2.5° increments. To adjust the desired angle, loosen the two wing nuts on the side. Change the angle and secure the Tilting base again with the wing nuts. (Fig. 40)

The tilt function is adjustable from 0° to 10°.

The 10° tilt function can be retrofitted. However, the assembly is quite complex and must be carried out very carefully by an instructed and qualified person! It is therefore recommended to use the pre-assembly option (can be selected when ordering).

The screw connections are provided with a screw lock. Therefore, the screws must be heated before loosening. Each screw connection must then be provided with a screw lock again (Loctite®).

An extension set can be used to adjust the adjustment range of the tilt function 10° to 12.5° - 20°. This extension set can be easily retrofitted.

You can also find detailed assembly instructions in videos on the Internet:
<http://www.thomashilfen.de/hercules-small-montage>

8.4 Additional head pad

The additional head padding provides an even softer contact surface for the head. The filling can be individually adjusted in the 3 chambers (middle and both sides) by removing filling material. Use the zip on the back of the pad for this.

For assembly, loosen the backrest cushion at the top of the backrest plate. Now pull the elastic band of the upholstery over the upper part of the backrest upholstery. Now pull the backrest cushion over the top of the backrest plate.

8.5 Headrest extension (sides)

This lateral extension of the headrest protects users who are sensitive to sensory overload when driving.

The extension is also suitable for users who need greater support on the sides. The extension can be adjusted individually per side

For assembly, first pull the cushion off the headrest (Fig. 41)

Mount the extension in the slots and tighten the screws again. (Fig. 42)

Now mount the cushion on the headrest again. For the correct fit, stick the Velcro strip supplied onto the headrest and attach the cushion to it.

Pull the original Velcro strip off the headrest to avoid damaging your vehicle.

8.6 Back extension (head area)

The back extension in the head area provides an enlarged area against which the user's head can rest. First pull the back cushion off the top of the back plate.

Use the two holes in the back plate to screw the back extension to the back plate. (Fig. 43) The slots in the backrest extension offer the possibility to adjust the height individually to the user. Tighten the nuts firmly.

To fit the back cushion, it is necessary to adjust the available length of the back cushion. To do this, loosen the lower end of the backrest cushion under the seat cushion. Now the backrest upholstery is pulled back over the extension of the backrest plate. Now the lower end of the backrest upholstery is fastened again under the seat upholstery on the Velcro strap.

8.7 Shoulder pads

The shoulder pads provide support in the shoulder area.

The two elastic straps are pulled over the side impact protection. One band is placed on the back and one band on the side.

8.8 Lateral trunk support, firm / swivelling

Lateral trunk support provide support for the upper body that can be individually adjusted to the user in height and width. The two pads are adjusted independently of each other, thus an optimal adaptation to the user is possible.

The lateral trunk support are mounted in the two slots of the back plate. (Fig. 44)

To mount the pads, first loosen the back padding at the top of the back plate. (Fig. 15)

Mount the lateral trunk support pads on the front of the back plate. (Fig. 45) Insert the screws through the back plate and the mounting bracket from behind. Fit the nuts from the front. Tighten the nuts firmly. Now the back cushion can be pulled over the top of the back plate again.

8.9 Armrests

The armrests provide a support surface for each arm of the user.

The armrests are mounted in the two slots in the back plate. (Fig. 46). Insert the screws through the back plate and the mounting bracket from behind. The nuts are mounted from the front. The armrests can be adjusted in height. Tighten the nuts firmly.

For easy positioning of the user, the armrests can be swivelled upwards. (Fig. 47)

When the user is safely positioned, the armrests are swung down again.

8.10 Table (only in combination with the armrests)

The table provides continuous, large-area support for the forearms First mount the armrests. Now position the table on the armrests. (Fig. 48).

Connect the table to the armrest on each side using the plug-in fastener. (Fig. 49)

The position of the table can now be adjusted using the lower straps. (Fig. 50) Then pull the elastic straps at the bottom of the front end of the table over the armrests on both sides.

8.11 Belt connector

The belt connector prevents the shoulder belts of the 5-point positioning belt from being pushed too far to the side. Thus, the shoulder straps remain in the optimal position over the user's shoulders.

The belt connectors are attached underneath the shoulder belt pads.

Tip: First mount the belt connector completely on one side before loosening the second side.

Release the lower part of the shoulder harness from the harness tensioner and pull out the harness. Now thread the harness connector onto the released harness. (Fig. 52)

Now thread the loose end of the harness back into the harness tensioner. Make sure that the harness connector is always positioned below the harness tensioner and that the harness is not twisted.

8.12 Abduction block

The abduction block supports the positioning of the legs and prevents the legs from crossing.

The holder for the abduction block is positioned on the top of the seat depth plate and screwed from below. (Fig. 53)

The abduction block is guided into the holder with the square tube and locked with the tommy screw. (Fig. 54)

If height adjustment is required, the set screw can be loosened with an Allen key. (Fig. 55) Now the square tube can be extended downwards. The threaded pin is tightened again. This gives you a larger height adjustment range.

8.13 Footrest, foldable

The footrest provides support for the user's feet. To get in and out, the footrest is folded up. A magnet secures this position of the footrest. (Fig. 56)

The footrest is pushed from the front into the receptacle under the seat plate. (Fig. 57) Adjust the depth of the footrest to the seat in the vehicle. Screw the footrest to the sides under the seat plate with the enclosed 2 Cap screws (1 per side). (Fig. 58)

The red eccentric levers on both sides can be used to set the desired angle. Make sure that the eccentric levers point downwards after the adjustment. (Fig. 59)

Select the height of the lower cross-connection so that it supports the Hercules Small securely in the footwell of the vehicle. The height can be adjusted using the side screws. (Fig. 60) Make sure that the two Allen screws are present on the footrest.

If the footrest is not at the appropriate height for the user, loosen the set screws of the footrest on both sides. Adjust to the appropriate height for the user and retighten the set screws. (Fig. 60)

8.14 Stand

The stand provides support for the Hercules Small in the footwell of the vehicle.

The stand is pushed from the front into the receptacle under the seat plate. (Fig. 57) Adjust the depth of the support foot to the seat in the vehicle. Screw the support leg to the sides under the seat plate with the enclosed 2 Cap screws (1 per side). (Fig. 58)

The red eccentric levers on both sides can be used to set the desired angle. Make sure that the eccentric levers point downwards after the adjustment. (Fig. 59)

Select the height of the lower cross-connection so that it supports the Hercules Small securely in the footwell of the vehicle. The height can be adjusted using the side screws. (Fig. 60)

8.15 Seatfix adapter

The Seatfix adapter ensures a firm connection with the vehicle.

Note: Prerequisite for the use of the Seatfix adapter is an existing ISOFIX equipment of the vehicle. This must be checked before purchase!

The Seatfix adapter is pushed from behind into the receptacle under the seat plate. (Fig. 61) Loosely screw the enclosed 4 cap screws (2 on each side) into the receptacle so that the Seatfix adapter can still be moved. (Fig. 62)

Determine the required position of the Seatfix adapter: To do this, place the seat in the vehicle and insert the Seatfix connectors (with yellow release) into the Isofix anchors of the vehicle. Pull to check that the connectors are securely engaged. The backrest of the Hercules Small must have as much contact as possible with the backrest of the vehicle. Mark the desired position at the junction of the Seatfix adapter and the receptacle under the seat plate.

Now loosen the Seatfix connectors and remove the Hercules Small from the vehicle again. Fix the Seatfix adapter in the marked position with the four cap screws.

8.16 Stabilizing bow

If your vehicle does not offer an Isofix connection, the stabilizing bow can be used as an alternative.

The stabilizing bow is pushed from behind into the receptacle under the seat plate. (Fig. 61) Loosely screw the enclosed 4 cap screws (2 on each side) into the receptacle so that the stabilizing bow can still be moved. (Fig. 63)

Determine the required position of the stabilizing bow: To do this, place the seat in the vehicle and guide the stabilizing bow between the seat surface and the backrest of the vehicle seat. The backrest of the Hercules Small must have as much contact as possible with the backrest of the vehicle. Mark the desired position at the junction of the stabilizing bow and the receptacle under the seat plate.

Take the Hercules Small out of the vehicle again and fix the stabilizing bow in the marked position.

The stabilizing bow can also be used upside down, depending on the vehicle seat. To do this, mount the bracket with the bend either upwards or downwards. (Fig. 63)

8.17 Incontinence seat cover

The incontinence seat cover is placed on the seat cushion (Fig. 64), then the small hook straps are attached to the back of the back cushion. The long strap is passed around the backrest and fastened in the plastic buckle. The seat cushion is now pushed forward and fastened under the seat cushion with the front hook strap.

It is possible to insert other commercially available absorbent pads into the incontinence seat pad to increase the absorption capacity if required.

8.18 Belt buckle safety pad

The belt buckle safety pad (Fig. 65) protects the belt buckle of the 5-point positioning harness from being accidentally opened/unbuckled by the user while driving. Open the 5-point harness and remove the crotch pad from the basic equipment. Now attach the belt buckle safety pad to the seat pad and insert the belt buckle. After closing the 5-point harness, insert the pin from behind through the eyelet of the harness pad and secure it by attaching the black magnetic lock at the front. To open, place the magnetic key on the black magnetic lock and remove it.

8.19 Front cover

The front cover (Fig. 66) is used when the vehicle is often exposed to high temperatures. In such cases, it provides additional protection for the child against the effects of high temperatures (burns). To install the front padding, lift the seat padding at the front edge, degrease the front 2 cm of the seat depth plate, and stick on the self-adhesive hook tape strip.

8.20 Chin support

The Chin support (Fig. 67) serves to stabilize the head during lateral or forward movement. The hook-and-loop fastener on the two chin pads is attached to the strap padding with Velcro, allowing it to be adjusted to the user.

The hook-and-loop fastener at the front of the chin connects the two chin pads at the front and can be adjusted and positioned correctly for the user.

The wedge supplied can be positioned under the Chin support using Velcro to ensure optimal support on the chest.

8.21 Second belt guide, top

A second upper belt guide can be fitted for flexible use on different car seats. (Fig. 6) The vehicle's 3-point safety belt can be used on the right or left side of the car seat without having to change the upper belt guide first. For more information on fitting, see "Changing the side upper belt guide".

8.22 Car seat protector with back

The car seat protector with back protects your car seat from damage or paint abrasion by the Hercules Small, which could possibly occur when using the swivel function or during driving.

First fit the car seat protector to your vehicle seat before fitting the Hercules Small in the vehicle.

8.23 Original fabric (for special construction)

If further support in special construction is required for the user, the original fabric can be requested.

9. Technical Data

Hercules Small	Dimensions in cm / inch
Seat depth	31 - 39 cm / 12.2 - 15.4"
Seat width	21 - 33 cm / 8.7 - 13"
Backrest height (seat inside)	57 - 80 cm / 22.4 - 31.5"
Back width (seat)	45 cm / 17.7"
Head height (seat - lower edge of headrest)	
(Min back height)	33,5 - 36,5 cm / 13.2 - 14.4"
(Max back height)	57 - 60 cm / 22.4 - 23.6"
Head width	11,5 - 20 cm / 4.5 - 7.9"
Shoulder height (belt guide)	33 - 60 cm / 13 - 23.6"
Shoulder width	45 cm / 17.7"
Shoulder width with shoulder pad	34,5 cm / 13.6"
Basin width	23 - 35 cm / 9.1 - 13.8"
Knee width	23 - 36 cm / 9.1 - 14.2"
Back angle (max.)	90° - 145°
Seat tilt (basic model)	0°
Seat tilt (with tilt plate 10°)	0° / 2,5° / 5° / 7,5° / 10°
Seat tilt (with tilt plate 20°)	10° / 12,5° / 15° / 17,5° / 20°
Lower leg length (with footrest)	16 - 42 cm / 6.3 - 16.5"
Height lateral trunk support (upper edge)	28 - 34 cm / 11 - 13.4"
Distance lateral trunk support	20 - 33 cm / 7.9 - 13"
Armrest height (upper edge)	15,5 - 31 cm / 6.1 - 12.2"
Armrests width	24 - 35 cm / 9.4 - 13.8"
Underarm support (top edge)	19,5 - 30 cm / 7.7 - 11.8"
Underarm support cut-out (depth)	11 - 23 cm / 4.3 - 9.1"
Underarm support cut-out width	30 cm / 11.8"
Abduction block (W x H x D)	7 x 10 x 10 cm / 2.8 x 3.9 x 3.9
Max. overall dimensions (W x H x D)	55 x 85 x 34 cm / 21.7 x 33.5 x 13.4"
min. Folding dimensions (W x H x D)	55 x 23 x 60 cm / 21.7 x 9.1 x 23.6"
Weight of the seat (basic model)	11,6 kg / 25.6 lbs
User height	100 - 150 cm / 40 - 59"
Max. user weight	18.1 - 50 kg / 40 - 110.2 lbs

10. Material specifications

Upholstery: Polyester

The following materials are used in the product: aluminum (body/accessories), steel (belt guide/accessories), PA6 (plastic handles), PU (upholstery), and polyester (covers).

All upholstery is made of high-quality foam, which is also used in pressure ulcer prevention and, in combination with the breathable Cover, ensures an optimal seating climate.

The upholstery material used is “flame retardant,” tested in accordance with DIN EN 1021-1 and DIN EN 1021-2.

The degree of resistance to ignition of materials and assemblies has been tested in accordance with standards and assessed as safe.


The 100% polyester fabric is a high-quality, breathable fabric and has been tested positively for biocompatibility. (DIN EN ISO 10993-5)

11. Cleaning, Maintenance, and Repair

11.1 Cleaning, Care Instructions, and Disinfection on a Daily Basis

11.1.1 Padding

The padding can be easily removed in a few simple steps and washed separately. The padding consists of a fabric cover and sometimes fillers made of foam. Check if the padding has a zipper or a hook and latch fastener. If it does, open the padding and remove the foam filler before washing the cover. Please also note the following care instructions.

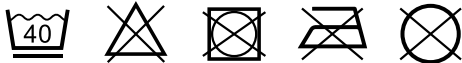We would like to point out that even high-quality materials can fade as a result of long and intense exposure to sunlight or frequent washing.

When changing users, a “padding set for reuse” is offered by the manufacturer.

11.1.2 Hook and Loop Fasteners

To keep the hook and loop fasteners working well, clean them now and then with a brush. Always close or cover the hook and loop fasteners before washing.

11.1.3 Plastic Parts

The plastic parts on the product can be cleaned and disinfected with standard soap (neutral soap).

11.1.4 Metal Parts

The metal parts on the product can be cleaned and disinfected with standard soap (neutral soap).

11.1.5 Disinfection

Clean the padding, plastic parts, and metal parts before disinfecting.

Damp wipe all the plastic and metal parts on the product with disinfectant. Only water-based products should be used. When disinfecting note the directions from the manufacturer in question.

The padding and the covers have to be disinfected before reusing. To do so use the right detergent in keeping with the guidelines of the Association of Applied Hygiene VAH (Verbund für Angewandte Hygiene e.V.) or use a "padding set for reuse" offered by the manufacturer.

11.2 Maintenance and inspection by the distributor

11.2.1 Preparation

A copy of the Instructions for Use has to be enclosed with the product. If one is not available, request a copy from the distributor. Familiarise yourself with how the product works. If you are not familiar with the product, read the Instructions for Use before testing it. Clean the product before starting to test it. Please note the care instructions in the Instructions for Use and product-specific test instructions. If you have further questions, please contact the staff member you dealt with or Thomashilfen directly.

Service and repairs to the product should only be done by qualified staff. It is the user's responsibility to ensure that the prescribed inspections / maintenance are done regularly and in a timely fashion. If damage is apparent, the user has to actively inform and have the damage fixed immediately by qualified staff.

11.2.2 Maintenance Schedule — Annual Inspection

	Area	Function		Damage		Replace	
1	Seat unit	👁 / ↻		👁 / ↻			
2	Angle adjustments	👁 / ↻		👁 / ↻			
3	Padding			👁		↻	
4	Belts	👁 / ↻		👁 / ↻			
5	Plastic parts	👁 / ↻		👁 / ↻			
6	Screws and nuts	👁 / ↻		👁 / ↻			
7	Labels			👁 / ↻			
Accessory							
8	Padding			👁		↻	
9	Adjustment options	👁 / ↻		👁 / ↻			
10	Plastic parts	👁 / ↻		👁 / ↻			
11	Metal parts	👁 / ↻		👁 / ↻			
12	Screws and nuts	👁 / ↻		👁 / ↻			
done by:				on:			
👁 = Inspection ↻ = Reuse ✓ = Test completed							

11.3 Spare Parts

Only original spare parts should be used for repairs. A list of the spare parts is available at www.thomashilfen.de. To receive the right spare parts or for possible technical recalls, we normally require the serial number / UDI-PI and the exact UDI-DI.

Removal / installation of spare parts should only be done by qualified staff! In the event of returns, please make sure the product is securely packaged for transportation.

11.4 Storage

Store the product in a safe place when it is not being used. Do not lay or put any heavy objects on the product while it is being stored. Protect the product from weather like rain, intense sunlight, and frost. If the product is wet, wipe the metal and plastic parts dry and let the padding dry before storing the product.

11.5 Disposal

If the product is no longer usable due to its condition or it has reached the end of its service life, it can be recycled at the local waste disposal company. Please refer to the local guidelines for recycling and disposal.

12. EU Declaration of Conformity

We hereby declare as the party solely responsible that this product and any versions and accessories comply with the basic regulations. The current EU Declaration of Conformity can be downloaded at www.thomashilfen.de or requested from us directly.

13. Warranty

Thomashilfen offers you the two-year warranty as required by law for this product. The warranty period starts when the item is handed over. The warranty includes all claims which impact function. Not included is damage which is the result of improper use (e.g. overloading) and natural wear and tear.

14. Period of Use

The product has a period of use of up to 6 years if used correctly and serviced regularly by a professional as per the information from the manufacturer. The product can be used beyond this point if it is in safe and technically sound condition. Time spent in storage at the distributor does not count towards the period of use.

The theoretical period of use does not equate to a guaranteed service life. It depends on the intensity of use and always undergoes an individual assessment by the distributor.

The period of use does not apply to wear parts like padding, covers, etc.

15. Service Life

The product has a service life of 8 years or 2 reuses (3 users) if the requirements stipulated for the period of use are met. The service life consists of the period of use and the storage times between reuses.

16. Reuse

The product is suitable to be reused. Before being passed on to the next user the product always has to be carefully cleaned and disinfected. The condition has to be checked by trained staff for any wear and damage. Any issues have to be remedied to ensure the continued safe use of the product. A copy of the Instructions for Use is a component of the product and has to be passed on to the new user.

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Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG